
# MEDICAL ASSISTANCE TOILET SEAT

  
TO VIEW THE  
INSTALLATION VIDEO VISIT:  
[www.centoco.com/resources-videos.php](http://www.centoco.com/resources-videos.php)  
OR SCAN QR CODE

INSTALL  
SEAT IN  
UPRIGHT  
POSITION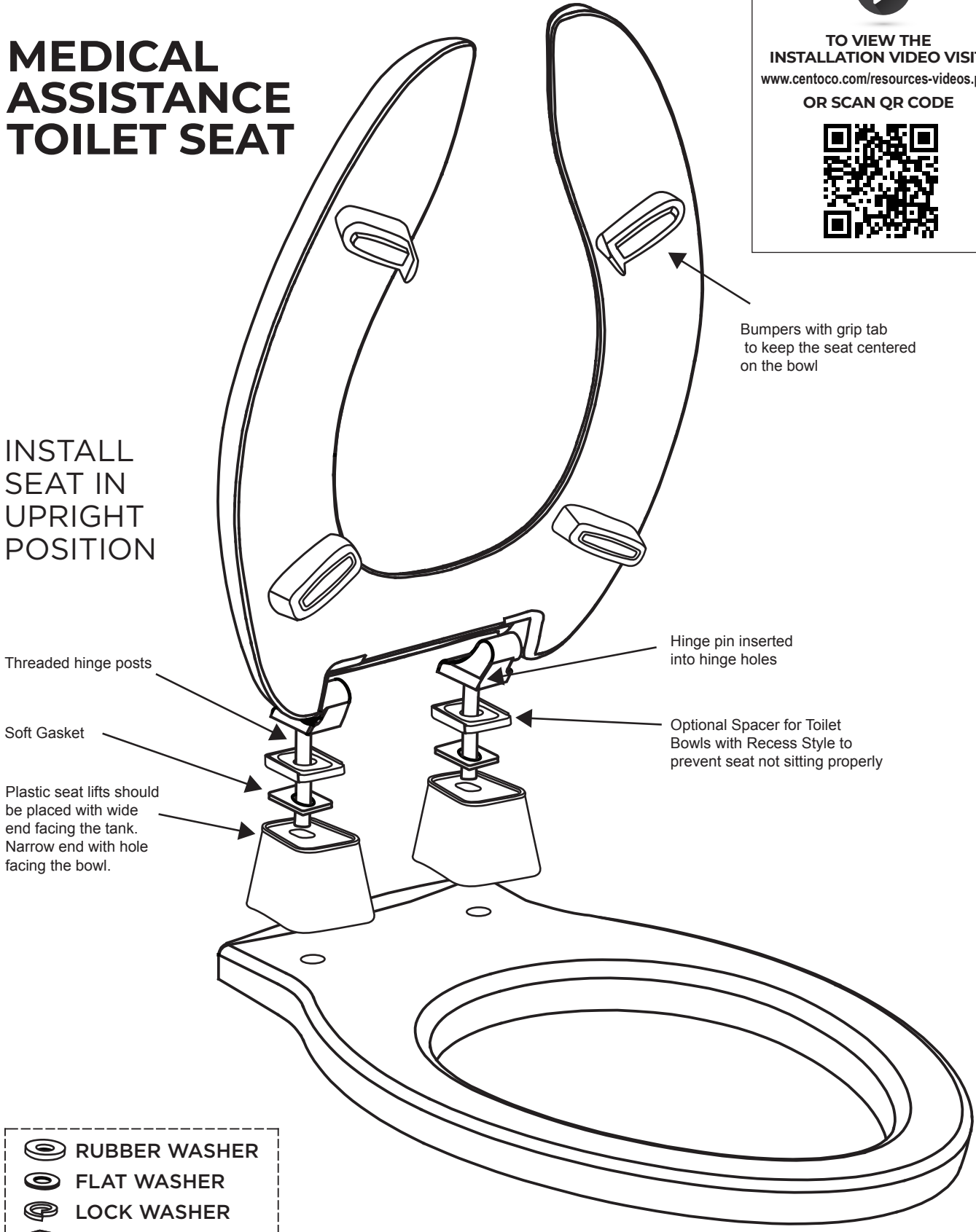

Bumpers with grip tab  
to keep the seat centered  
on the bowl

Threaded hinge posts

Soft Gasket

Plastic seat lifts should  
be placed with wide  
end facing the tank.  
Narrow end with hole  
facing the bowl.

Hinge pin inserted  
into hinge holes

Optional Spacer for Toilet  
Bowls with Recess Style to  
prevent seat not sitting properly

-  RUBBER WASHER
-  FLAT WASHER
-  LOCK WASHER
-  HEX NUT

# Medical Assistance Toilet Seat

Centoco medical assistance toilet seats are ADA compliant and designed to assist those with disabilities or limited mobility. Once installed these seats are elevated 2" - 3" allowing for less pressure on the knees. A higher toilet seat can also assist with the transfer from a wheelchair to the toilet by raising the toilet seat to a level closer to that of the wheel chair.

## INSTALLATION INSTRUCTIONS:

**NOTE: INSTRUCTIONAL VIDEO AVAILABLE ON YOUTUBE AND WWW.CENTOCO.COM**

Tools: you will need a 9/16" or adjustable wrench to complete the installation

1. Insert the hinge pins into the hinge holes of the toilet seat.
2. Place the optional spacer if required and soft gasket on the threaded hinge posts.
3. Place the plastic seat lifts into position over the bolt holes on the toilet.
4. Holding the hinge pins and hardware in place, put the seat in an upright position and slide the hinge posts through the holes in the plastic lifts and toilet bolt holes.
5. From the underside of the bowl place the soft round washer, metal round washer and metal lock washer on the hinge post and start to thread the bolt on manually. Finish tightening with the 9/16" or adjustable wrench until firmly secure.
6. Repeat on the opposite side.

Note that the inside bumpers may not touch the rim of the bowl depending on the model For maintenance we recommended periodically checking and re-tightening the nuts Please clean the seat with a soft, damp cloth and only use a mild soap.

To learn more about Centoco products, visit our new website at [centoco.com](http://centoco.com)

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Centoco Manufacturing LLC: 809 E. Fifth Street, McCrory, Arkansas, USA, 72101
Centoco Plastics Ltd.: 2450 Central Avenue, Windsor, Ontario, Canada N8W 4J3